




Short-term patient stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.



**1** Remove white cover to expose patch side of iFIX Patch Adapter.

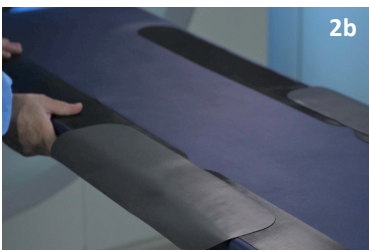

**2** Attach the loop side of the iFIX Patch Adapter to existing (standard) hook fastener on both sides of patient table.  
Attach additional iFIX Patch Adapters and/or cut iFIX Patch Adapters if required.



**3** Attach iFIX Fleece or Head Mask to iFIX Patch Adapter.  
See respective Instructions for Use for iFIX Fleece or Head Mask.



**4** Clean and disinfect all re-usable anchor products after use. Residues of iFIX Fleece can be removed using commercially available lint roller.  
Make sure there is complete wetting of the anchor surfaces with disinfection agent.  
Recommended disinfection agent: alcoholic surface disinfectant (e. g. Incidin Liquid or Mikrozyd AF Liquid).

iSYS REF	Product Name	Specifications
16179L001	iFIX Patch Adapter 50	4 Patch Adapters per box, width 20 cm, length 50 cm

## WARNINGS AND PRECAUTIONS

- Non-Sterile
- Latex Free and Hypoallergenic
- Safe for Use in MRI